

2-in-1 LEG RELIEF WEDGE

INSTRUCTIONS FOR USE

Remove the wedge from the packaging and unfold the wedge.

Pull the valve out of its recessed position and remove the cap from the valve. Insert the tip of the pump into the valve (Fig. A).

Pinch the valve lightly with your thumb and forefinger. Use the palm of the same hand to steady the tip of the pump. Use the pump to push air into the wedge until inflated to the desired fullness. **NOTE:** Inflating the wedge 80% full is ideal for most people. When 80% full, pushing the wedge with your finger should leave a moderate indentation. A completely inflated wedge is too firm for most people.

Remove the pump from the valve. Place the cap on the valve and push the valve into the wedge.

TO DEFLATE THE WEDGE, pull on the exposed tab to release the valve from the recessed position within the wedge. Remove the cap from the valve. Pinch the valve opening together with two fingers to release the air.

TO CLEAN, use a damp cloth to wipe away any dirt or spilled liquids.

SPOT CLEAN WATER ONLY AIR DRY ONLY







⚠ IMPORTANT SAFEGUARDS

- NOT A TOY. Only use as instructed.
- Do not sit, stand, or jump on the unit.
- Do not over-inflate.
- · Only use the air pump that is included.



30-Day Money-Back Guarantee

Trend Makers is so confident that you will enjoy your product, that we cover every product with our 30-day Money Back Guarantee. However, if you decide that you don't love our product, your order is covered by our 30-day Money Back Guarantee which starts the day your product is delivered. You have 30 days to return or replace your product. Proof of purchase is required. Monetary refunds do not include original processing and handling fees. We do not price match.

Please contact us at customercare@trendmakersinc.com with any questions.

IF YOU DID NOT PURCHASE DIRECTLY FROM TREND MAKERS:

Please follow their return policy.



Distributed by: Trend Makers, LLC Fort Walton Beach, FL 32547 Made in China © 2022 Trend Makers, LLC

